

Quick Clean™

Micro-Steam Sterilization Bags

- Use Drinking water quality for the following described cleaning procedures
- After each use thoroughly wash the breast shields, valve and membrane. Rinse immediately after use in cold water, then wash thoroughly in warm soapy water in a disposable bowl, making sure all breastmilk is removed. Rinse under running water. Dry thoroughly with a clean paper towel.

•DO NOT PLACE THE TUBING IN WATER



1. Immediately before next use put the 2 breast shields along with the valves and membranes into the steam sterilisation bag.



2. Add 60ml of water to the bag.

Using Less Than 60 ml water may warp the parts



3. Seal the bag tightly. Put into microwave on the centre of the turntable, ensure the bag is upright, and the turntable is able to rotate without obstruction.

Microwave should be 800w power rating



4. Heat in the microwave on full power for **3** minutes

Heating for more than the recommended time may disfigure the plastic parts

5. **Leave to stand for 2 minutes to cool down.**

Taking care, as the contents may still be **hot**, remove the bag from the microwave.

6. Before opening the bag, remove the water by pouring out through steam vent



7. Remove parts from the bag and dry them with a clean paper towel. Assemble for immediate use.

8. Dry the inside of the Microwave with a clean paper towel, and leave the microwave open to allow it to dry between use